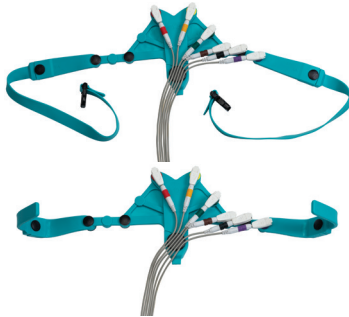
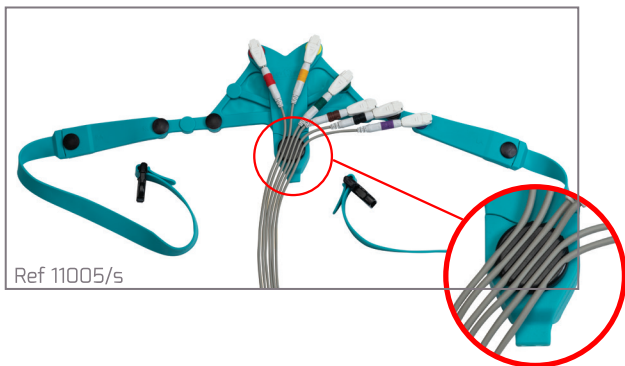
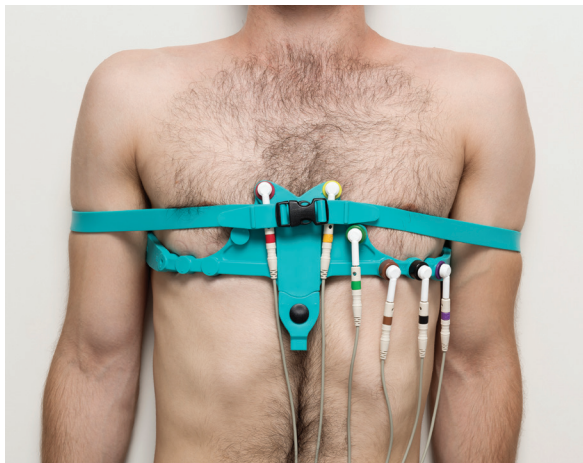
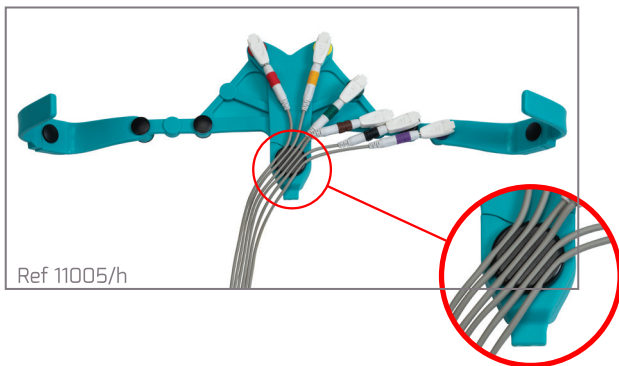
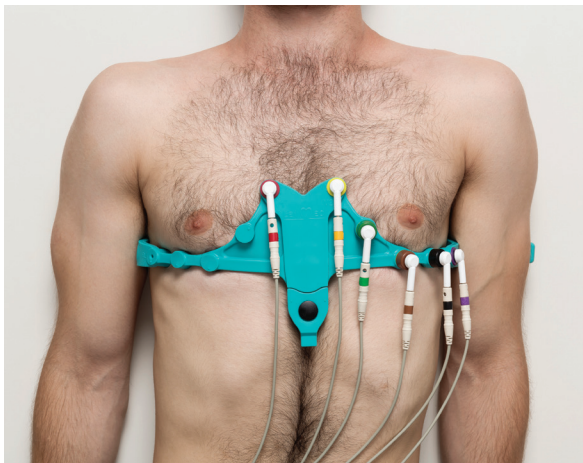
LevMed **AllBrand™** ECG Electrode Belt
Ceinture d'électrodes ECG **AllBrand™** LevMed
Cinturón de Electrodo ECG LevMed **AllBrand™**
Cintura con elettrodi ECG **AllBrand™** di LevMed
EKG-Elektrodengürtel LevMed **AllBrand™**
LevMed **AllBrand™** ECG-elektroderiem
LevMed **AllBrand™** ECG Elektróda Öv
LevMed **AllBrand™** EKG Elektrot Kemer
Ζώνη Ηλεκτροδίων ΗΚΓ LevMed **AllBrand™**
LevMed **AllBrand™** حزام إلكترود تخطيط القلب

O Produto: Cinta Elétrodo ECG, da Empresa, LevMed **AllBrand™**
LevMed **AllBrand™** EKG elektrodoový pás

INSTRUCTIONS FOR USE

LevMed AllBrand™ ECG Electrode Belt

with Handles (Cat# 11005/h)

with Straps (Cat# 11005/s)

Rx Only

Caution: Read the complete list of instructions before using the product

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician

INTENDED USE: LevMed Allbrand™ ECG Electrode Belt is a reusable accessory intended to be used with ECG devices for short-term 12-lead rest ECG recordings for adults and pediatrics.

WARNINGS AND CAUTIONS

- Do not use this device for any purpose other than the stated intended use.
- Remove device before performing defibrillation.
- LevMed ECG Belt is intended to be used for rest ECG recordings, make sure the patient is relaxed, lying still, on a flat surface.
- Stay away from devices that create strong electromagnetic fields.

CONTRAINDICATIONS

- Do not use LevMed ECG belt on irritated or damaged skin.

SYMBOL LEGENDS

Caution, refer to instructions for use

SN

Serial number

REF

Catalogue number

Manufacturer

LOT

Date of manufacture

Non Sterile

WEEE- Waste of electrical and electronic equipment

LevMed AllBrand™ ECG Electrode Belt

INSTRUCTIONS FOR USE

1. DEVICE DESCRIPTION

LevMed AllBrand™ ECG Belt is a system of ECG electrodes cast in a hyper-flexible silicone that is similar to ECG V1-V6 pre-cordial electrodes. The Belt is flexible and can be stretched to fit all body shapes and sizes, men and women alike; when fitted to the chest, it stretches to the required proportions, and the distance between the electrodes is automatically set. The Belt can be connected to any standard ECG patient cable.

2. APPLICATION INSTRUCTIONS

Scan the applicable QR code for the Application process of the AllBrand™ ECG Belt:

For AllBrand with Handles (Cat# 11005/h) For AllBrand with Straps (Cat# 11005/s)

3. CLEANING & DISINFECTION

Wipe the LevMed ECG Belt with disinfectant wipe after each patient. Use only disinfectants that are dedicated for use on medical devices.

Caution: Do not use disinfectants containing petrol.

WARRANTY

LevMed Ltd. warranty applies to the silicone components of the ECG Belt for a period of 12 months from the date of purchase or 15 months from the date of production, whichever is shorter. Cables, internal wires, electrodes, plastic parts, and other parts integrated within the ECG Belt are covered by warranty for a period of 3 months from the date of purchase or 6 months from the date of production, whichever is shorter. Warranty does not apply to any accessories supplied.

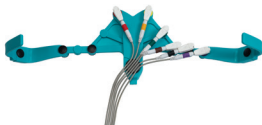
This warranty does not include or cover malfunctions caused by unreasonable use, noncompliance with user and maintenance instructions, or damage caused by unauthorized or unqualified repairs.

ATTENTION! Do not use if the electrodes are not in full contact with the body.

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Caution, refer to instructions for use

SN

Serial number

REF

Catalogue number

Manufacturer

LOT

Date of manufacture

Non Sterile

WEEE - Waste of electrical and electronic equipment

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Kinesis Medical B.V.